

February 20, 2023

This letter is to provide our customers with a recommended cleaning agent for the **Hydrocollator® Heating Units** and the **ColPac® C-5 Chilling Unit**. This recommendation provides cleaning instructions for the both the interior and the exterior of the Hydrocollator® and is based on cleaning agents that will not harm the stainless-steel surfaces.

There are two options in cleaning the **interior surface** of the heating unit and chilling unit.

1. Use a lightly damp, non-abrasive 70% isopropyl alcohol, 30% water wipe. Rinse thoroughly with water after cleaning.
2. To dissolve away deposits, use a 20% vinegar, 80% water solution. This solution is safe for cleaning the interior if a non-abrasive cloth is used. Rinse thoroughly with water after cleaning.

For the exterior surface of the Hydrocollator®, use a stainless-steel polish and a soft cloth only. Many polishes are available from various supplier. Do not use any cleaner for the exterior finish of the Hydrocollator® except a stainless-steel polish. Abrasive cleaners and scouring pads can damage the finish of the unit.

•
•